

- NEW Collection Devise for OVA & PARASITE AND STOOL C & S
- NEW Collection Devise for STOOL CULTURE & SENSITIVITY
- Zipper Pouches for Frozen Specimens

Dear Valued Client:

Genesis Clinical Laboratory recently changed collection devices from the 2-vial pink and gray Para Pak to the **1-vial Remel YELLOW SAF Fixative for Ova and Parasites** and from a YELLOW cap vial to a **GREEN cap vial with red liquid for Stool Cultures**.

Ova and Parasites **Test Code #6300** **1-Vial Remel YELLOW SAF Fixative**

Collect: Stool. Preserve stool collection in Remel Yellow Cap SAF Fixative within one hour of collection. Recommended collection: three separate stool specimens within a 10-day period. An individual order must be submitted for each specimen.

Storage/Transport Temperature: Preserved Stool 20-25°C (Room Temp)
Remarks: Indicate suspected parasites. Specimens analyzed to determine the efficacy of treatment should be collected three to four weeks after completion of therapy. Vials for preservation are available from Genesis Supply Department. Antibiotics may affect results of exam.

Unacceptable Conditions: Dry specimens, leaking containers, specimens contaminated with oil, barium or urine, multiple specimens (more than one in 24 hours), unpreserved specimens, rectal swabs.

Stability – Preserved Stool

- Ambient: 9 months
- Refrigerated: 9 months
- Frozen: Unacceptable

Stool Culture and Sensitivity **Test Code #6045** **1-Vial Remel Dark Green Cap Cary Blair (with red liquid).**

Collect: Stool. Preserve in Remel Dark Green Cap Cary Blair Vial (with red liquid) immediately. 6

Storage/Transport Temperature: Stool in Cary Blair Media (Genesis Supply) at 2-8°C (Refrigerated)

Unacceptable Conditions: Non-sterile or leaking containers, multiple specimens (more than one in 24 hours), dry specimens, delayed transport without use of appropriate preservative.

Stability

- Room Temperature: 4 days
- Refrigerated: 4 days
- Frozen: Unacceptable

NOTE: Please deplete current inventory of O & P and Stool Culture Vials. As you order new supplies, you will receive the new vials.

Zipper Pouches for Frozen Specimens

Just a reminder, our easy-to-use [Zipper Pouches for Frozen Specimens](#) with instructions are available for sending Genesis your frozen specimens. These pouches are reusable. Couriers will remove samples and requisitions from the zipper pouch, placing the specimens on dry ice and return the zipper pouch for reuse to your lockbox.

Your service representative can answer specific questions on how to transfer samples and use zipper pouches during their visits to your facilities. For immediate assistance, please contact your sales/service representative or client service at **708-783-3280, press 2.**

P.S. Don't forget to check expiration dates on your supplies!

Client: _____				Genesis Clinical Lab			
Address: _____				3231 S. Euclid Ave.			
City: _____		Rt: _____		Berwyn, IL 60402			
Acct # _____		Date _____		PH-708-783-0011 --- Fax 708-783-5724			
REQUISITIONS				PRINTER PAPER			
GENERAL LAB REQ _____							
CYTOLOGY LAB REQ _____				PRINTER PAPER LASER - BLUE _____ REAM			
SURGICAL PATH REQ _____							
SPECIALIZED LAB REQ _____				PRINTER PAPER LASER- YELLOW _____ REAM			
QTY	ITEM #	DESCRIPTION	QTY	ITEM #	DESCRIPTION		
BLOOD COLLECTION TUBES			CULTURE & TRANSPORT				
PK	6820	Lt Blue Top -Sod Citrate 2.7ml (3083)	EA	9410	Blood Culture Bactec Aerobic		
EA	8638	Dark Blue Top /Trace Elements 6ml (8380)	EA	9411	Blood Culture Bactec Anaerobic		
EA	8639	Dark Blue Top/K2/EDTA 6ml (8381)	EA	7535	Cary Blair Media (1 Vial, Green Cap)		
EA	8686	Brown Top Tube/K2/EDTA 3ml (7855)	EA	6988	Ova & Parasite Kit (1 Vial, Yellow Cap)		
PK	6831	Gray Top /Sod Fluoride/EDTA 2ml (7587)	EA		Aptima Urine Collection Kit 50/Bx		
EA	12139	Green Top-Lithium Hep 6ml (7886)	EA		Aptima Unisex Swab Collection Kit 50/Bx		
EA	17485	Green Top-Sodium Hep 6ml (7878)	EA	0177	Culture Swab BBL Routine 50/Bg		
PK	6856	Microtainer SST 50/PK (5967)	EA	0175	Anaerobic Specimen Collector (236500)		
PK	6858	Microtainer Lavender 50/PK (5973)	EA	99-08011	VCM Transport Medium - Cervical / Female		
PK	6855	Lavender Top K2-EDTA 4ml (7861)	EA	99-08014	VCM Transport Medium - Urethral / Male		
PK	6827	Gold Top SST 3.5ml (7983)	EA	99-08013	VCM Transport Medium - Lesion/Other		
PK		Gold Top SST 5.0ml (7986)	EA	99-08015	VCM Transport Medium - Nasopharyngeal		
PK	6829	Red/Gray Top SST 8.5ml Plastic (7988)	EA		EZ Polyfoam Flu Swab (220144)		
EA		PPT Plasma Prep Tube- EDTA 5.0ml (2788)	EA	12115	Hemocult II Slide (61100)		
PK	6825	Red Top Plain Chemistry Tube 10ml (7820)	EA				
PK	120007	Pink Top -K2 EDTA 6ml Blood Bank (7899)	EA				
EA	12134	Yellow Top ACD "A" 8.5ml (4606)				COLLECTION & TRANSPORT	
EA		Yellow Top ACD "B" 6ml (4816)	EA	8536	Cup Urine Sterile 90ml White Cap100/Bg		
KT	T05900301	Quantiferon - TB Blood Collection Tubes	EA	4954	Cup Urine C&S Blue Cap w Tube 50/Bx		
BX		NEEDLES & PHLEBOTOMY	EA	4979	Urine Vacutainer Tube Yellow Top Plain 10ml		
BX	0016	Alcohol Prep Pads 200/Bx	EA	4992	Urine Vacutainer Tube Red/Yel Top 8ml		
BX	12099	Antiseptic Towelettes 100/Bx	EA	4966	Urine Transfer Straws		
BX	0274	Band-Aid Cloth 1x3 100/Bx	EA	4953	Urine Culture Tubes 4953 50/Bx		
BX	7091	Band-Aid Pediatric 100/Bx	PK	RK5	Urine Cup Non Sterile 100/Pk		
EA	1597	Biohazard Container Sml 1Qt PSC ONLY	EA	80024	24 Hour Urine Container		
EA	0187	Biohazard Container Med 1/2 Gal PSC ONLY	PK	62.554.02	Urine Transport "Bullet" 50/Pk		
EA	9140	Biohazard Container Lrg 2Gal PSC ONLY	EA	0182	Stool Container 8OZ		
BX	0041	Cotton Balls 130/Bx	PK	7480	Specimen Transport Bags 100/PK		
BX	6866	Butterfly 21x3/4" 50/Bx	EA	8674	Serum Transfer Tube & Cap for Frozen		
BX	6857	Butterfly 23x3/4" 50/Bx	EA		Color Coded Caps (circle) RED LAV BLUE GRN		
BX	20469	Needle 21x1.25 Safety 50/Bx				CYTO & HISTO	
BX	20468	Needle 22x1.25 Safety 50/Bx	EA	7470	30 ml Biopsy Container Prefilled 50/Bx		
PK	0857	Gauze Pad 2x2" 12 Ply 200/Pk	EA	7895	90 ml Biopsy Container Prefilled 50/Bx		
EA	6830	Needle Holder Adult	PK	8786	Cyto Brush 100/Pk		
EA	6832	Tourniquet Non Latex	EA	7266	Cytolyt Vials 30 ml 20/Tray		
		LABELS	EA	8387	Cytology Spray Fixative		
RL	8642	Frozen Specimen	BX	10465	Pap Kit Standard 25/BX		
RL	7295	Specimen Tube Label	BX	12044	Pap Kit w Brush 25/Bx		
RL	105664	Dymo Lab Link Label	PK	13046	Thin Prep w Spatula & Brush 25/Pk		
RL		STAT Specimen Labels	PK	13046	Thin Prep w Brooms 25/Pk		
EA		PRINTER CARTRIDGES			MISCELLANEOUS		
EA	E250A11A	E250D Toner Cartridge Lexmark 250/350 & 6350	PK	8585	Glucola 50G Orange 6/Pk		
EA	E260A11A	E260D Toner Cartridge Lexmark 260/360 & 6360	PK	19815	Glucola 75G Fruit Punch 6/Pk		
EA	12a7400	6210 Toner Cartridge Lexmark 321/323	PK	8244	Glucola 100G Lemon Lime 6/Pk		
EA	12a7410	8210 Toner Cartridge Lexmark Key	EA	7960	Centrifuge 6 place		
EA	C7115A	HP 15A Toner Cartridge HP1200	EA	13337	Patient Log Book		